

# All-in-One wound management system:



## PowerFlex® AFD ABSORBENT FOAM DRESSING

- Foam pad and PowerFlex cohesive in one
- Quick and easy to apply
- Absorbs and holds up to tens times its weight in fluid
- Strong, flexible PowerFlex base
- Controlled compression - will not constrict
- Straight, hand-tear - no scissors needed
- Effective on puncture wounds, abrasions and lacerations
- Conforms to wound bed - reduces maceration and promotes healing
- Won't stick to wound bed



### PowerFlex AFD Construction



- 1 PowerFlex Layer - Strong & flexible, sticks to itself for easy wrapping
- 2 Waterproof Film - Prevents leakage and contamination
- 3 Absorbent Foam - Patented hydrophilic foam pad draws and holds fluid


Wrapping a Leg Wound:



After cleaning the wound, unroll to expose absorbent foam pad.



Place foam pad over wound.

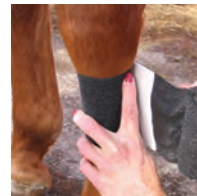

Cover wound with foam pad & wrap bandage around so it covers first layer.



Prior to each wrap around leg, pull bandage out approx. 6".



Continue to wrap at desired tension with 50% overlap.




Seal end of bandage to bottom layer with fingernail for a strong, solid seal.

Wrapping a Neck Wound:



Once the wound has been cleaned, unroll to expose the absorbent foam pad. Place the pad over the wound.

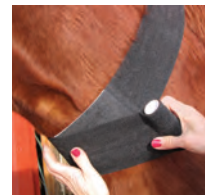

Wrap bandage around, covering the first layer. Continue wrapping; seal layers of bandage with fingernail (as shown above).

Photographed at Ever after Mustang Rescue Biddeford, Maine  
www.mustangrescue.org

### PowerFlex® AFD Ordering Information...

Catalog #	Size	Rolls/Case	
3744BK-008	4" x 2.5 yds.	8	Black
3744S-008 (sterile)	4" x 2.5 yds.	8	White