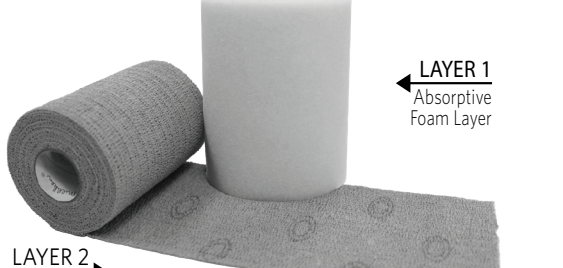
COFLEX TLC LS
TWO-LAYER LONG STRETCH COMPRESSION KIT
Instructions for Use

Two-Layer Long Stretch Compression
Dvoudvrstvá komprese s vysokou roztažitelností
To-lags kompres med lang strekk
2-Schicht lange strecke komprimierung

Two-Layer Long Stretch Compression
Dvoudvrstvá komprese s vysokou roztažitelností
To-lags kompres med lang strekk
2-Schicht lange strecke komprimierung
Komprimierung zu 2-lagige Kompression
2-lags kompresjon med lang strekk
Dwuwarstwowy system uciskowy dlugiego natężenia
Compresso de duas camadas de longa extensão
Двухслойная компрессия большой растяжимости

Two-Layer Long Stretch Compression
Dvoudvrstvá komprese s vysokou roztažitelností
To-lags kompres med lang strekk
2-Schicht lange strecke komprimierung
Komprimierung zu 2-lagige Kompression
2-lags kompresjon med lang strekk
Dwuwarstwowy system uciskowy dlugiego natężenia
Compresso de duas camadas de longa extensão
Двухслойная компрессия большой растяжимости

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Continue to spiral wrap up the leg, covering Layer 1 with 50% overlap while maintaining even tension (Figure 7).
Cut or hand-trim excess material or tape to keep the bandage in place (Figure 8).

Removal/Disposal
إزالة المنتج من
Geltung des Produkts pro your facilities protocol.

Storage
الحفظ
Best results, store CoFLEX TLC LS (Standard & Lite) in its original packaging at room temperature...

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CoFLEX TLC LS (Standard & Lite) is a two-step non-latex compression system. Layer 1 is a foam roll, with an odor and itch control properties, that provides comfort and absorbs moisture...

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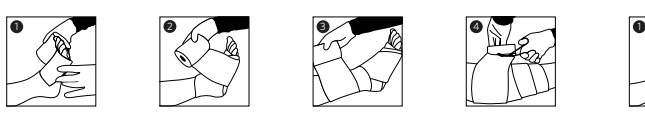
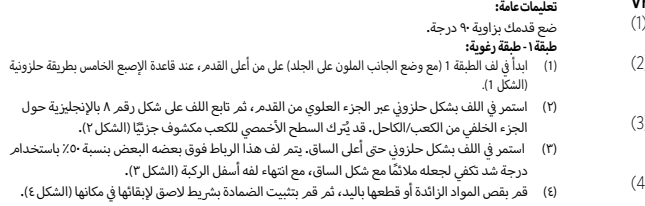
CoFLEX TLC LS (Standard & Lite) is designed to treat venous leg ulcers and other related conditions.

الاحتياطات والتحذيرات
Precautions & Warnings
Should be used under the care of a licensed healthcare professional.

Follow appropriate protocol for the wound condition before applying.
CoFLEX TLC LS (Standard & Lite) Layer 2 may initially be applied with less tension allowing the patient to adjust...

تعليمات الاستخدام
General Instructions:
Position the foot at a 90° angle.

Begin wrapping Layer 2 on the top of the foot, at the base of the fifth digit in a spiral technique (Figure 5).

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Continue to spiral wrap up the leg, covering Layer 1 with 50% overlap while maintaining even tension (Figure 7).

Removal/Disposal
إزالة المنتج من
Geltung des Produkts pro your facilities protocol.

Storage
الحفظ
Best results, store CoFLEX TLC LS (Standard & Lite) in its original packaging at room temperature...

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