

Instructions for Use (Read carefully before using the device)

Descriptions

Every Milliken Healthcare Products, LLC TRITEC™ dressing utilizes Active Fluid Management® technology created by Milliken Healthcare Products. This three part design combines two layers of high performance fabric using a micro-knit process. This creates a powerful moisture transfer mechanism that pulls away and traps excess exudate. The combination of these fabrics keeps the periwound skin dry, the wound bed moist, and harmful exudate away from a healing wound.

Directions for Use

TRITEC dressings can be used throughout the healing process. Prepare the wound bed according to clinical protocol such as debridement and/or cleaning and rinsing with sterile saline or water. Remove the dressing from the package and apply to a highly exudating wound with the printed side of the dressing away from the wound. Optimal performance of TRITEC dressings requires direct contact with the wound. If required, TRITEC dressings can be cut to various shapes and sizes. TRITEC dressings can be secured with an appropriate secondary dressing such as roll gauze, adhesive border gauze, composite island dressing, transparent film or tape. TRITEC dressings can be used under compression. For dry wounds you can place the dressing print side towards the wound with a source of fluid behind to add moisture to the wound.

Directions for Removal

TRITEC dressings should be changed when clinically indicated or as prescribed by a healthcare professional. Carefully remove the fixative and the dressing from the wound. Normally the dressing does not adhere to the wound. If adherence is observed, the dressing can be moistened with sterile saline or water to ease removal



healthcare.milliken.com
medproducts@milliken.com

920 Milliken Road
Spartanburg, SC 29303 USA
Tel: 866-491-6556
8 am - 5 pm E.S.T.

MILLIKEN HEALTHCARE PRODUCTS, LLC

The Milliken logo, featuring the word "Milliken" in a stylized, blue, cursive script font. A small "TM" trademark symbol is located at the end of the word.

Made in U.S.A.
© 2014 Milliken Healthcare Products, LLC. All rights reserved. TRITEC™, Active Fluid Management®
and AFM® are trademarks of Milliken Healthcare Products, LLC.
Document #TritecPr2

Indications

The TRITEC™ dressings are indicated for the management of acute and chronic wounds: partial thickness burns, incisions, skin grafts, donor sites, lacerations, abrasions, and Stage I-IV dermal ulcers (vascular, venous, pressure and diabetic).

Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner (U.S.A. only).

Precautions and Observations

TRITEC dressings are not approved for use directly on full thickness burns.

The Active Fluid Management® performance of TRITEC dressings may be impaired by use with petrolatum based ointments.

Storage

TRITEC dressings should be stored at temperatures less than 131°F. Keep dry

Sterilization

TRITEC dressings are sterile as provided. Do not use TRITEC dressings if the primary package is visibly damaged or broken. Do not resterilize TRITEC dressings. For single use only.