


MILLIKEN HEALTHCARE PRODUCTS, LLC

ASSIST SILVER ABSORBENT DRESSING



DRESSING ARCHITECTURE & INDICATIONS FOR USE

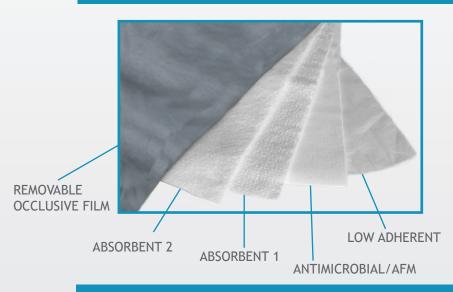

ASSIST Silver Absorbent is indicated for the management of acute and chronic wounds:

- Partial thickness burns
- Incisions
- Skin grafts
- Donor sites
- Lacerations
- Abrasions
- Stage I-IV dermal ulcers: vascular, venous, pressure, and diabetic
- Lymphedema

BENEFITS OF ASSIST SILVER ABSORBENT DRESSING

- Eliminates need for non-adherent layer
- Saves O.R. time during application
- Effective sustained release of ionic silver for up to seven days¹
- Minimal maintenance reduces nursing time
- Active Fluid Management (AFMTM) Technology is designed to pull and trap excess exudate
- Each dressing can hold up to 1,000cc of fluid at saturation¹
- May be cut to shape or size
- Grey film can be removed to allow for air flow
- Cost effective



INSTRUCTIONS FOR USE

- Place with the low-adherent layer directly against the wound (grey side away from the patient)
- Affix with elastic netting, abdominal belt, roll gauze, stapling to itself, or other common securement methods
- Change the dressing when saturated or soiled
- Remove grey layer if air flow is desired

APPLICATION EXAMPLES

DONOR SITES



SKIN GRAFTS



SJS/TEN ULCERS



PRODUCT INFORMATION

ASSIST Silver Graft Cover Dressings			
Product Code	Size	Ea/Bx	Ea/Cs
3000051176	8" x 20"	10	30
3000051175	4" x 48"	N/A	10
3000084768	6" x 6"	10	100
ASSIST Silver Absorbent Dressings			
3000117008	Extremity	N/A	15
3000117007	Vest	N/A	10
3000117009	Half Bed	N/A	5
3000116362	18" x 24"	N/A	40
3000006964	24" x 36"	N/A	20
ASSIST Absorbent Dressings			
3000004622	24" x 36"	N/A	25

Milliken Healthcare Products, LLC